- ⓘ Istruzioni di montaggio, manutenzione, uso e garanzia
- Ⓒ Assembly, maintenance, use instructions and guarantee
- Ⓔ Instrucciones de montaje, mantenimiento, uso y garantía
- Ⓓ Montage-, Wartungs-, Gebrauchs- und Garantieanleitungen
- Ⓕ Instructions destinées au montage, à l'entretien, à l'usage et à la garantie
- Ⓖ Instructies van montage, onderhoud, gebruik en garantie

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Fixing kits examples

1: Concrete, aerated concrete or brick

Proposed solution: use of wooden screws and universal dowels.

In case of aerated concrete use of gas concrete dowels.

2: Gypsum plaster board and plate for fixing work

Proposed solution: use for hollow dowels.

3: Gypsum plaster board and plate for fixing work

Proposed solution: in order to obtain a higher stability of the light wall structure, either use a reinforced strutting or backfill the wall.

Warning

The pack does not contain the fixing anchors to the wall, we recommend choosing them carefully in function to the type of wall.

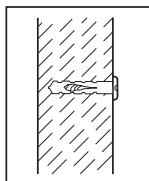
For optimum fixing, we recommend using at least 3 anchor screws Ø4,5-6 and blocks SX8 in holes at 120° and 4 anchor screws Ø6-8 and blocks SX10 for each washer/plate.

Please check carefully the wall structure solidity (underground). If necessary, use special dowels. To solve possible doubts please contact a professional or supplier of fastening system.


The declared maximum load capacity is only guaranteed with optimum fitting.

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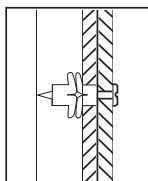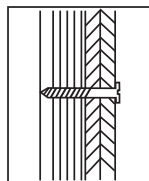
Piece description

Goman safety accessories are made of :

- a) Ø 32 steel tube with anti-corrosion treatment and are coated with wear resistant material. They are designed to support a static load at the most unfavourable point of at least 100 Kg. These values correspond to a load exerted by a person weighing a maximum of 150 Kg.
- b) aluminum tube coated with 2 mm. nylon tube, total Ø 35, non toxic, U.V. resistant material. They are designed to support a static load at the most unfavourable point of at least 80 Kg. These values correspond to a load exerted by a person weighing a maximum of 120 Kg.

Use instructions

Handles: used as a support and rest for persons: wall mounting.

Bars: used as a support and rest for persons: tilting up and down action (descent with clutch braking): wall or floor mounting

Small seats: used for sitting: tilting up action: wall installation.

Maintenance and cleaning instructions

Do not use detergents containing acids, alkalis, or chlorine or ones with abrasive products. Do not use abrasive objects (brushes for example).

Do not use preparations for the bathroom that can damage plastic materials.

We recommend cleaning the surface with water and a neutral detergent; rinse carefully and dry with a clean cloth.

If cleaning is performed in ways other than those specified above, the surface may be damaged. Damage due to unsuitable treatments are excluded from the guarantee.

Important warnings:

Exceeding the maximum load values may cause permanent product damage. If this happens, it is indispensable to suspend product use until checked by a specialised company commissioned for maintenance.

Improper use, such as for gymnastic exercises or as a step for reaching high-up objects, may cause accidents with serious consequences.

Good operation and product safety largely depend on the way the products are installed.

For this reason, fitting must be performed by qualified personnel.

Incorrect installation can seriously compromise product functionality.

CE conformity declaration

The product contained in this package fully satisfies the regulations in force (DM 14 June 1989 no. 236 and DPR 24 July 1996 no. 503)

and corresponds to Directive 93/42/CEE and successive modifications.

Guarantee

The guarantee conditions of our stockists in the different countries are valid for this product. Contact them in case of need.

Goman declines all responsibility for defective installation and improper use.

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Goman S.r.l.